

InfraLight Neck Loop – Instructions

1. Charge the battery fitted in the neck loop receiver continuously for 24 hours before first use. Please refer to your instruction booklet supplied with your *InfraLight* transmitter for further details.
2. Make sure that the transmitter is properly connected to your TV as per the transmitter instruction booklet.
3. The neck loop receiver is switched “off” when the rotary volume control is at position “0”. It is switched “on” when at position “1”.
N.B. Always switch to “off” when not in use, otherwise the battery will discharge.
4. Turn your hearing aid to position “T” or “MT” as appropriate for your hearing aid.
5. Adjust the volume control between numbers 1 and 5 to find a comfortable sound level – number 1 is the lowest, number 5 is the highest.

When you are not using your *InfraLight* system place the neck loop, switched off, in the charging recess on the top of your transmitter. In this way the battery will recharge ready for the next time you wish to use the neck loop.