



Operation

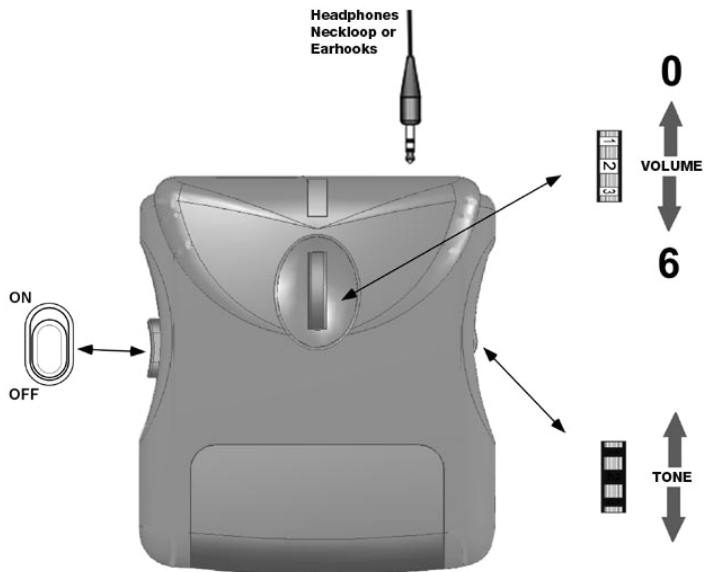

To recharge this system simply drop the receiver onto the charging slot of the Transmitter.

Please ensure that the receiver unit is switched off during charging or it will not reach its maximum charge capacity.

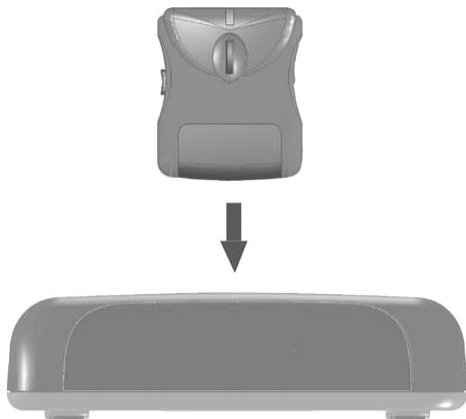

IMPORTANT

Please remember to switch your hearing aid to the "T" Loop position if you are using this system with a neckloop or earhooks.




Sonumaxx Quick Start Guide

Battery Installation and charging


Batteries for this receiver unit are already installed, however they will require a long charge of at least 12-14 hours before operation.





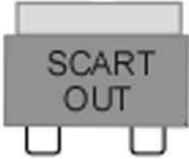

Should you need to change or remove these batteries please follow the procedure below.

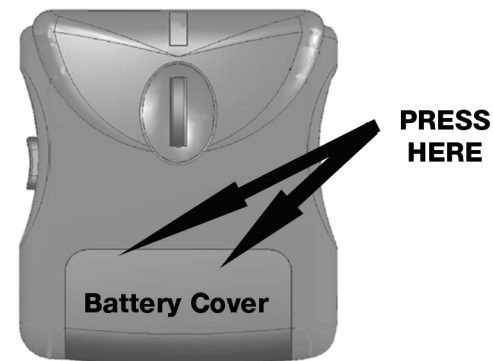Step 1

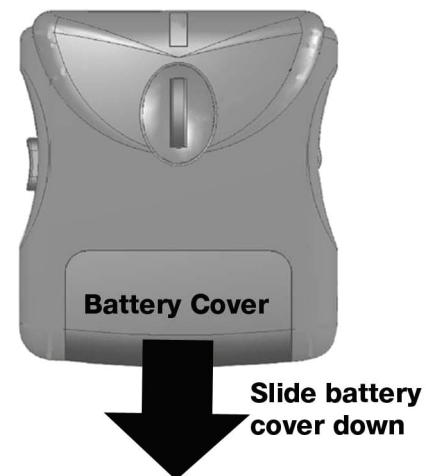


Step 2

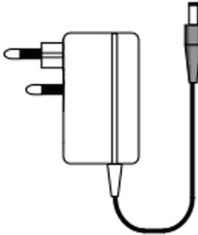

EQUIPMENT LIST	
Transmitter	
Receiver	
Mains Power Supply	
Phono - Phono Lead	
Scart adaptor	
Phono Socket—3.5mm Plug Lead	

EASY SETUP

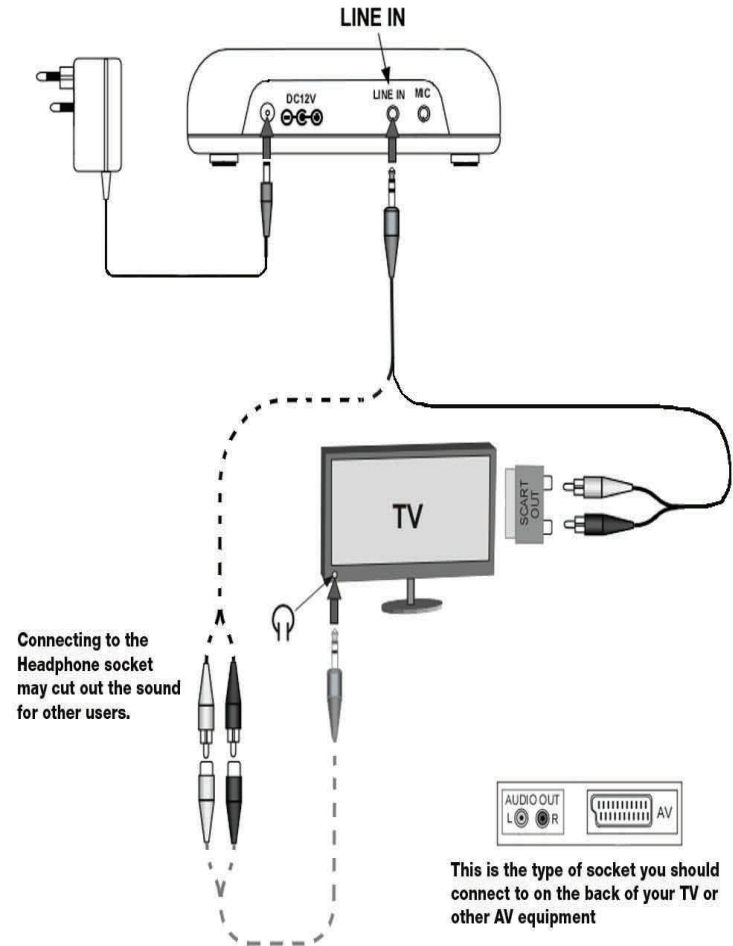

Connecting to your equipment using the method shown above should give excellent results, however not all Televisions and Audio equipment are the same.

If you connect to the TV/Audio equipment via the headphone socket the sound may be cut off for other users, however some of the newer types of television have a setting where you may be able to override this (refer to your user guide)

The preferred method of connection is through the Audio Output L & R sockets, this should give you the correct sound in any situation.